trophon® EPR and trophon®2

Compatible Ultrasound Probes List

The probes on this list have been approved for use with the trophon® EPR and / or trophon®2 following extensive compatibility testing performed in conjunction with the Original Equipment Manufacturers (OEMs).

Approved probes are listed by manufacturer, series (Series), if applicable and then model (Model) number in the tables below.

Please check your probe(s) compatibility with the trophon EPR and trophon2, indicated by: 'Y'

Special Conditions:

Model types labelled with symbols have special conditions when used with the trophon EPR and trophon2.

Please take note of these special conditions when using these probes.


- ¹ Mindray (Zonare) (E9-4): trophon EPR and trophon2 are approved for use with the E9-4 Transducer running software version 4.7.323L or higher on the z.one ultra Ultrasound System and all software versions on the ZS3 Ultrasound System. Running software versions prior to 4.7.323L on the z.one ultra after using trophon EPR or trophon2 disinfection system can result in acoustic output levels exceeding FDA limits.

Note: Verify the software version on the z.one ultra by pressing the service key.
- ² Prosonic (L5-10/40EPN): Only probes manufactured after December 30 2011 are compatible.
- ³ Ultrasonix (L14-5/38): Only transducers with the serial number format TRA1.1-L145RP.xxx are compatible. If uncertain, confirm with your Ultrasonix representative.
- ⁴ Hitachi Aloka Medical: Where the serial number is specified in the list with brackets e.g. (M00621~), only probes with these serial numbers are compatible for use for this probe model.
- ⁵ These probes contain lumens that cannot be high level disinfected in trophon2/trophon EPR. For disinfection of the lumens, refer to the probe original equipment manufacturers' IFU for reprocessing information. In USA, Canada and Japan, trophon2/trophon EPR can be used to high level disinfect the external surfaces of these probes only.
- **CPP** The trophon CPP (Curved Probe Positioner) is an accessory to the trophon EPR to improve the positioning of approved curved probes in the device chamber. Model types that require this accessory are indicated by: 'Y' in the 'CPP' column of the list. Refer to the trophon CPP Instructions for Use for more information on setup and use.

Note: trophon CPP is NOT required for the trophon2.
- **T** The trophon EPR T-Clamp (see Figure 1 below) is available to fit Ultrasound Probe cables ranging from Ø5mm to Ø11mm. The standard probe clamp provided with trophon EPR fits probes with cable diameters from 8mm. Please contact your trophon EPR representative for further information, or to request a T-Clamp. Model types that require the T-Clamp are indicated by: 'Y' in the 'T Clamp' column of the list.

Note: T-Clamp is NOT required for the trophon2.
- **IPP** The trophon2 Integrated Probe Positioner (IPP) is a fully integrated mechanism in the trophon2 (see Figure 2 below) that improves the positioning of all approved curved probes in the trophon2

chamber. Model types that may require engagement of this mechanism are indicated by: 'Y' in the 'IPP' column of the list. Refer to the trophon2 User Manual for information on how to use the IPP.

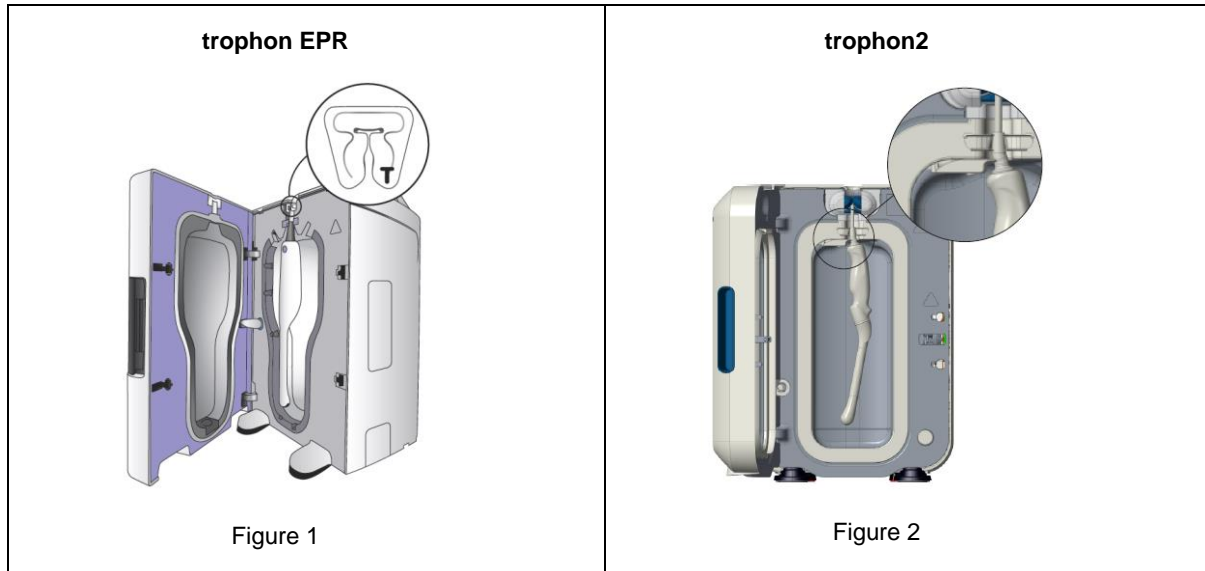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Canon Medical Systems Corporation (Toshiba)