

Cleaning and drying steps in high level disinfection (HLD) cycle



READY-TO-USE trophon Companion Wipes simplify cleaning and drying steps to promote efficient HLD workflows.

STEP 1



Use the **Cleaning Wipes** to remove protein, gel, microbial load and other soil from probe. Wipe until visibly clean.

STEP 2



Low level disinfect (LLD) the probe cable and plug.

STEP 3



Dry probe with a **Drying Wipe** and load into trophon for HLD.

STEP 4



During room turnover, use the **Cleaning Wipes** to low level disinfect the trophon start button and handle, ultrasound monitors, keyboards and probe holders.

STEP 5



Remove probe from trophon. Wipe with a **Drying Wipe** then store in a clean cover.

Note: A new wipe is recommended for each cleaning and low level disinfection step.