trophon[®]

Sonex-HL[®]

Only for use with the trophon[®] EPR and trophon[®]2

 Nanosonics Limited
14 Mars Road, Lane Cove
NSW 2066, Australia
+61 2 8063 1600
info@nanosonics.com.au
www.nanosonics.com.au

USA Contact:
Nanosonics, Inc.
7205 E. 87th Street
Indianapolis, Indiana 46256
support@nanosonics.us
www.nanosonics.us

USA/ Canada Customer Support:
1-844-876-7466




MADE IN AUSTRALIA

E Cat: E8350MC

REF N05002

L03000 7.0 03/2020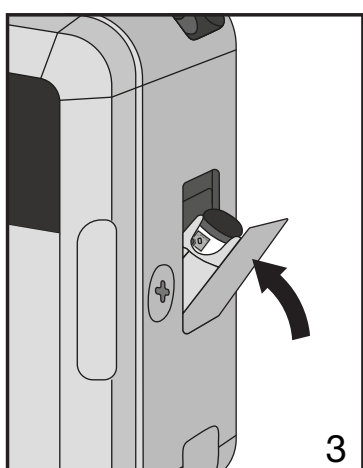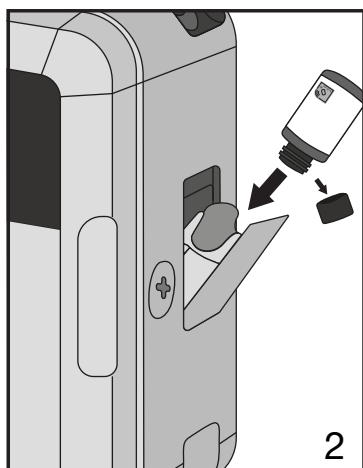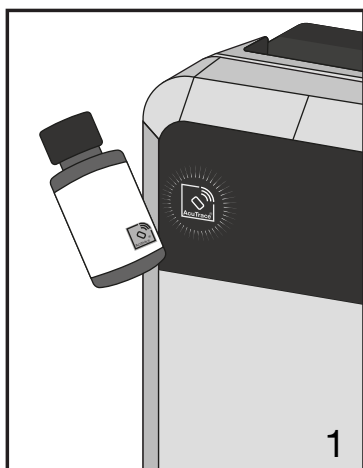

EN Instructions for Use

trophon[®] Sonex-HL[®]

Single use disinfectant delivery cartridge, containing multiple doses of disinfectant.

The following must be read in conjunction with your trophon[®] EPR / trophon[®]2 User Manual. This cartridge is specifically designed for use with the trophon EPR / trophon2 to deliver the correct dosage of disinfectant to ensure high level disinfection (HLD).

Each cartridge contains 80mL of 35% w/w hydrogen peroxide solution. Use of disinfectants with differing composition or concentration to that specified may result in damage to the trophon EPR / trophon2, probe and/or reduced effectiveness of the HLD process.

Refer to your trophon EPR / trophon2 User Manual to enable the AcuTrace[®] Technology, if available.

1. Warnings (Refer to the Safety Data Sheet for more information)

IMPORTANT: Cartridges are pierced at the top and the bottom side upon insertion of a new cartridge into the trophon EPR / trophon2. A small amount of disinfectant may remain in the cartridge, even when it has been fully used. Follow the instructions carefully to avoid injury.

- Always wear disposable gloves when handling disinfectant cartridges.
- Avoid contact with eyes and skin.
- Do not manually pierce the cartridge.
- Do not attempt to open or use a damaged or distorted cartridge.
- Do not attempt to refill or reprocess the empty cartridge after it has been removed from the trophon EPR / trophon2. This could lead to personnel injury and/or equipment/device malfunction or damage.
- Strong Oxidizer: Contact with other materials may cause a fire.
- Corrosive: Harmful if swallowed or inhaled. May cause eye and skin burns. Prolonged or repeated skin contact may cause dermatitis.

2. Emergency Guidelines

- **Spillage:** Always wear personal protective equipment. Turn OFF the trophon EPR / trophon2 at the power switch and at the mains power switch. Dilute with plenty of water and discard according to Occupational Health and Safety Guidelines for your institution.
- **Eye contact:** Rinse immediately with copious amounts of cold water, including under the eyelids for at least 15 minutes. Keep eyes open and do NOT rub eyes.
- **Skin contact:** Flush skin with copious amounts of soap and water for at least 15 minutes. Remove contaminated clothing and shoes. Consult a physician if irritation persists.
- **Ingestion:** Do NOT induce vomiting. If victim is conscious and alert, rinse mouth thoroughly with water. Never give anything by mouth to an unconscious person. Call a poison control centre.
- **Inhalation:** Remove person to fresh air. If symptoms persist, call a physician.

3. Warning Symbol Definitions

	Caution		Corrosive - 8
	Single Use		Oxidiser - 5.1
	No Air Transport		Wear Gloves

4. Other Symbol Definitions

	Batch No.		Consult Instructions for Use
	Product No.		Expires
	AcuTrace [®] RFID Zone		Remove the cap from the cartridge

5. Inserting the Disinfectant Cartridge

IMPORTANT: Do NOT use a distorted cartridge. Always check the expiration date on the cartridge before use.

 **Cartridge door will automatically open when the cartridge needs replacing. Do NOT force the cartridge door open as this may damage and render the trophon EPR / trophon2 non functional.**

- If AcuTrace is enabled, scan the AcuTrace RFID Zone on the cartridge label (figure 1).
- Remove the cap from the cartridge
- Place the cartridge NECK FIRST into the holder. Do NOT force the cartridge into the holder (figure 2).
- When situated in place correctly, the bottom of the cartridge should be in line with the top of the holder. The door should close smoothly. Do NOT force the door to close.
- The trophon EPR / trophon2 is now ready for use. Once the door is closed, it will automatically lock and cannot be reopened until the cartridge is consumed or purged.

6. Cartridge Removal

Cartridge removal is required when the cartridge has expired or is empty. This will be indicated by a screen message. The trophon EPR / trophon2 will need to be purged before being transported or when a cartridge has expired to ensure all disinfectant is removed. Refer to the trophon EPR / trophon2 User Manual for further information on purging.

- Follow the trophon EPR / trophon2 onscreen instructions. Wear gloves, the door will open automatically. When directed, the empty cartridge can be lifted out. Avoid touching the pierced areas of the cartridge.
- Do NOT shake or change the orientation of the cartridge. For empty cartridge disposal see section 7 below.
- Replace with a new cartridge. Refer to section 5 above.

7. Disposal of Used Cartridge

Follow procedures of your institution for the disposal of CORROSIVE or OXIDIZING materials or dispose of as per local environmental and government regulatory requirements.

8. Storage

	Temperature limits: (14°F – 77°F) (-10°C – 25°C)		Keep away from sunlight		Store this way up as indicated on cartridge packaging
---	--	--	-------------------------	---	---

- Store cartridges in ALL original packaging until use.
- Store cartridge upright as indicated on cartridge packaging.
- Cartridges may distort or explode if not properly stored.

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